


TECHNICAL MANUAL

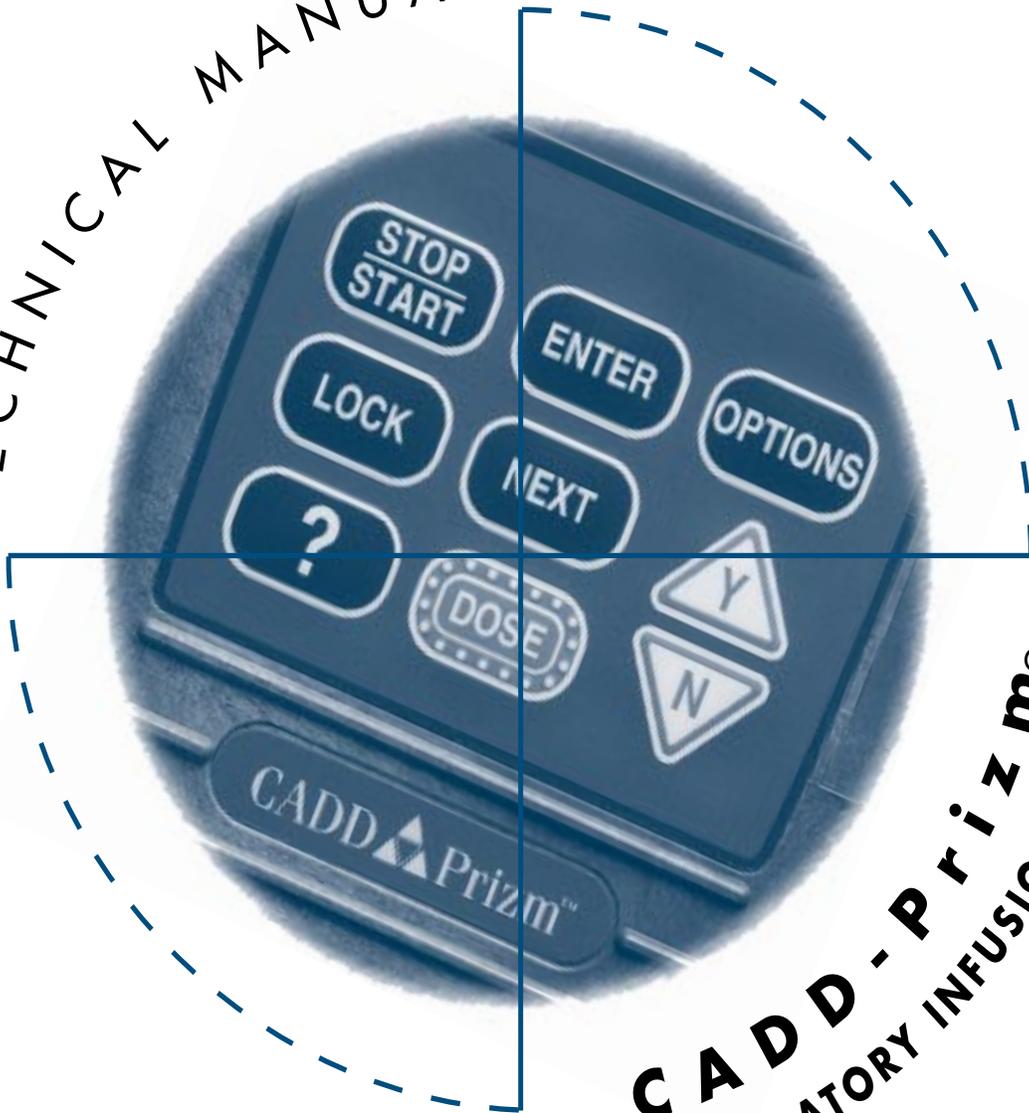

**CADD-Prizm®**  
AMBULATORY INFUSION PUMP

MODEL 6101

MODEL 6100

**Deltec**

This online version differs from the printed version. Certain information that is not intended for patients has been removed.

---

For detailed instructions, specifications, warnings, warranties and additional information on operating CADD® pumps, please refer to the *Operator's Manual* supplied with the product. If you have additional comments or questions concerning the operation of CADD® pumps, please call this number: (800) 426-2448. Our staff is available to help you 24 hours a day with the programming and operation of CADD® pump infusion systems.

*The issue date of this Technical Manual is included for the user's information. In the event one year has elapsed between the issue date and product use, the user should contact Deltec, Inc. to see if a later revision of this manual is available.*

***Issue Date: April 2002***

---

# Table of Contents

1. Introduction .....	1	6. Safety Features and Fault Detection .....	17
Limited Warranty .....	1	Hardware Safety Features .....	17
Exposure to Radiation or Magnetic Resonance Imaging (MRI) .....	1	Watchdog Timer Circuit .....	17
2. CADD-Prizm® Pump .....	2	Motor Driver/Motor Watchdog Circuit .....	18
Delivery Modes .....	2	Cassette ‘Type’ Sensor Circuit .....	18
Specifications (Nominal) .....	5	Latch/Lock Sensor Circuit .....	18
3. Batteries .....	8	Voltage Detector Circuit .....	18
Battery Compatibility .....	8	Software Safety Features .....	19
DURACELL® Alkaline Battery Life .....	8	7. Hardware and Software Fault Detection ..	20
ULTRALIFE® Lithium Battery Life .....	8	Overview .....	20
4. Construction .....	11	Order of Error Code Events .....	20
5. Theory of Operation .....	12	8. Cleaning and Inspection Procedures .....	21
Keyboard Circuitry .....	12	Inspection Recommendation .....	21
Data Memory EEPROM .....	12	Cleaning .....	21
Battery Backed RAM .....	12	Visual Inspection .....	21
Time Base Circuitry .....	12	Mechanical Inspection .....	22
LCD Circuitry .....	12	9. Testing Procedures .....	23
LED Status Indicators .....	13	Functional Testing .....	23
Flash PROM Technology .....	13	Air Detector Test (if Applicable).....	26
Gate Array Circuitry .....	13	Occlusion Tests .....	26
Audible Alarm Circuitry .....	13	Accuracy Testing .....	28
Watchdog Timer Circuit .....	13	Cleaning and Functional Testing Checklist ...	31
Motor Driver/Motor Watchdog Circuit .....	13		
Power Circuitry .....	14		
Voltage Reference Circuit .....	14		
Pumping Mechanism .....	14		
Pumping Characteristics .....	15		
Air Detector .....	16		
Upstream Occlusion Sensor .....	16		

# 1 Introduction

The Technical Manual is intended to provide a basic, but limited, understanding of the mechanical and electrical operation of the Deltec CADD-Prizm® Computerized Ambulatory Drug Delivery pump to persons familiar with this device. The CADD-Prizm® *Operator's Manual* should be used in conjunction with this publication for complete information.

This manual also outlines cleaning and functional testing procedures that can be performed on the CADD-Prizm® pump.

This technical manual is applicable to both the CADD-Prizm® PCS and CADD-Prizm® VIP pumps. For information on the CADD-Prizm® PCS pump refer to the information regarding the CADD-Prizm® VIP PCA application in this manual. Testing procedures in the back of this manual are intended to be performed with the pump in the PCA delivery mode.

## IMPORTANT NOTICE:

CADD-Prizm® pump operations and safety features are based on a microcomputer design. Inadequate servicing or tampering with the safety features of the pump may seriously affect performance and safety.

For that reason, **ALL SERVICING AND REPAIR OF THE CADD-Prizm® PUMP MUST BE PERFORMED BY DELTEC OR ITS AUTHORIZED AGENTS.**

The manufacturer's warranty agreement shall become null and void if the pump is not used in accordance with the *Operator's Manual* and *Instructions for Use* for the pump accessories; or, the pump is serviced by persons other than Deltec or those authorized by Deltec.

---

## Limited Warranty

The limited warranty associated with the CADD-Prizm® pump can be found in the product literature supplied with the product when originally purchased, which is incorporated herein by reference. **DELTEC SPECIFICALLY DISCLAIMS ANY OTHER WARRANTY, WHETHER EXPRESS, IMPLIED OR STATUTORY, INCLUDING, WITHOUT LIMITATION, ANY IMPLIED WARRANTY**

**OF MERCHANTABILITY OR FITNESS FOR USE.** Deltec further disclaims responsibility for the suitability of the system for a particular medical treatment or for any medical complications resulting from the use of the system. The manufacturer shall not be responsible for any incidental damages or consequential damages to property, loss of profits, or loss of use caused by any defect or malfunction of the system.

If you wish to receive additional information about the extent of the warranty on these products, please contact your Deltec representative or call Customer Service at (800) 426-2448.

All recommendations, information and literature supplied by Deltec with respect to the CADD® product line are believed to be accurate and reliable, but do not constitute warranties. No agent, representative, or employee of Deltec has authority to bind Deltec to any representation or warranty, expressed or implied.

---

## Exposure to Radiation or Magnetic Resonance Imaging (MRI)

---

### CAUTIONS:

(1) The pump **SHOULD NOT BE DIRECTLY IRRADIATED** by therapeutic levels of ionizing radiation because of the risk of permanent damage to the pump's electronic circuitry. The best procedure to follow is to remove the pump from the patient during therapeutic radiation sessions or diagnostic levels of radiographic and fluoroscopic radiation. If the pump must remain in the vicinity during a diagnostic or therapy session, it should be shielded, and its ability to function properly should be confirmed following treatment.

(2) Magnetic fields produced by magnetic resonance imaging (MRI) equipment may adversely affect the operation of the pump. Remove the pump from the patient during MRI procedures and keep it at a safe distance from magnetic energy.

## 2 CADD-Prizm® Pump

### Delivery Modes

The Deltec CADD-Prizm® pump provides measured drug therapy to patients in hospital or outpatient settings. The CADD-Prizm® pump is indicated for intravenous, intra-arterial, subcutaneous, intraperitoneal, epidural space or subarachnoid space infusion. Epidural administration is limited to short-term infusion of anesthetics and either long- or short-term infusion of analgesics. Subarachnoid administration is limited to short-term infusion of analgesics. The pump's flexibility

allows it to be used in a variety of settings. The CADD-Prizm® VIP (Variable Infusion Profile) pump may be programmed to deliver medication in one of four delivery modes: (1) PCA (patient-controlled analgesia) (2) Continuous (3) Intermittent and (4) TPN (total parenteral nutrition) (See Figures 2 through 5). The CADD-Prizm® PCS (Pain Control System) pump may be programmed to provide patient-controlled analgesia therapy. Figure 1 shows a diagram of the CADD-Prizm® pump.

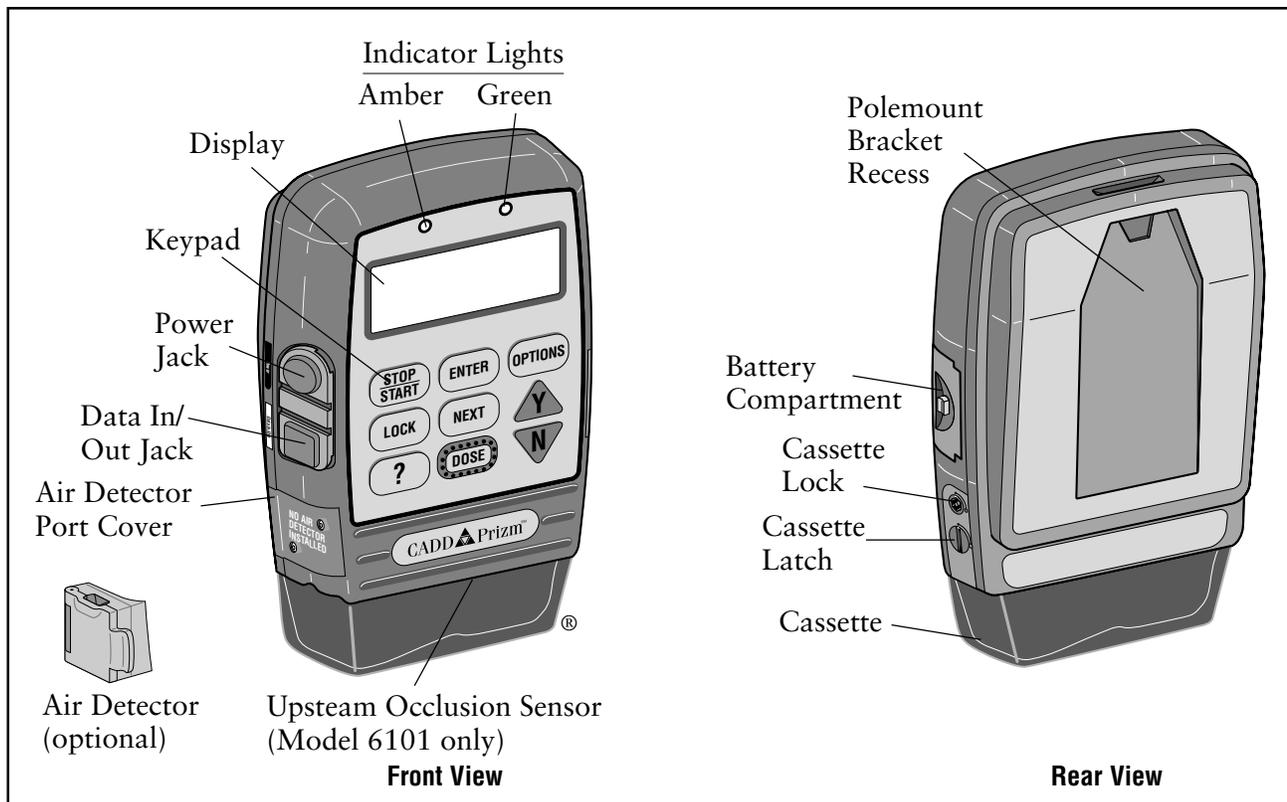


Figure 1. Front and back views of the CADD-Prizm® pump.

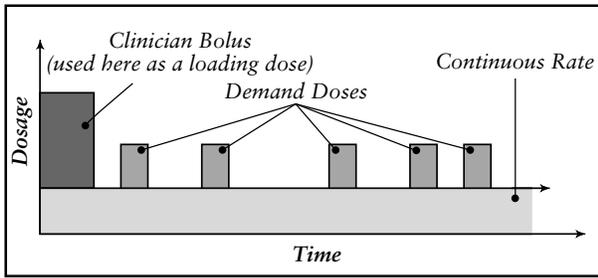


Figure 2. PCA mode delivery profile.

### PCA Delivery Profile

The PCA (patient-controlled analgesia) delivery mode is used for therapies that require a continuous rate of infusion, patient-controlled demand doses or both, such as patient-controlled analgesia.

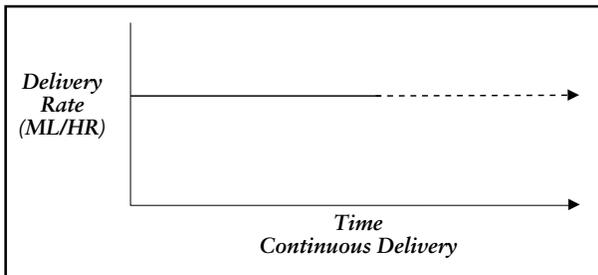


Figure 3. Continuous mode delivery profile.

### Continuous Mode Delivery Profile

The Continuous delivery mode allows the infusion of drug at a constant, programmed rate.

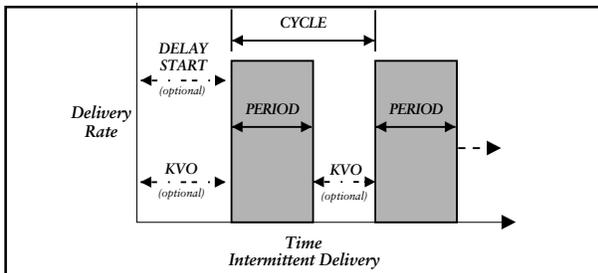


Figure 4. Intermittent mode delivery profile.

### Intermittent Mode Delivery Profile

The Intermittent delivery mode allows the infusion of a specific volume of drug at a regular programmed interval.

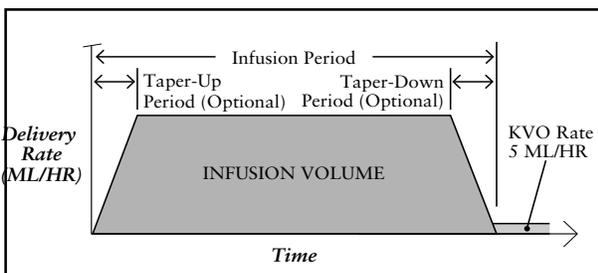


Figure 5. TPN mode delivery profile.

### TPN Mode Delivery Profile

The TPN (total parenteral nutrition) delivery mode allows the infusion of nutritional solutions or other fluids, with optional tapering at the beginning and end of infusion, and an optional KVO.

## PCA Delivery Mode Scroll Ranges

Units	Starting	Increment	Maximum	
Milliliters	0.10	<b>0.10</b>	<b>30.00</b>	
Milligrams & Micrograms	10% of concentration	<b>Mg only:</b> Values between 0.01 and 0.5: <b>Mcg only:</b> Values between 0.1 and 0.5: Values between 0.5 and 100: Values between 100 and 1000: Values greater than 1000:	0.01 0.1 0.1 1.0 10.0	<b>Concentration x 30</b>

Table 1. PCA delivery mode: continuous rate scroll ranges.

Concentration mg/ml	Milligrams			
	Demand Dose		Clinician Bolus	
	increment	max.	increment	max.
<b>0.1</b>	0.01	0.99	0.01	2
<b>0.2</b>	0.02	1.98	0.02	4
<b>0.3</b>	0.03	2.97	0.03	6
<b>0.4</b>	0.04	3.96	0.04	8
<b>0.5</b>	0.05	4.95	0.05	10
<b>1</b>	0.05	9.9	0.05	20
<b>2</b>	0.10	19.8	1.10	40
<b>3</b>	0.15	29.7	1.15	60
<b>4</b>	0.20	39.6	0.20	80
<b>5</b>	0.25	49.5	0.25	100
<b>10</b>	0.50	99.0	0.50	200
<b>15</b>	0.75	148.5	0.75	300
<b>20</b>	1.00	198.0	1.00	400
<b>25</b>	1.25	247.5	1.25	500
<b>30</b>	1.50	297.0	1.50	600
<b>35</b>	1.75	346.5	1.75	700
<b>40</b>	2.00	396.0	2.00	800
<b>45</b>	2.25	445.5	2.25	900
<b>50</b>	2.50	495.0	2.50	1000
<b>55</b>	2.75	544.5	2.75	1100
<b>60</b>	3.00	594.0	3.00	1200
<b>65</b>	3.25	643.5	3.25	1300
<b>70</b>	3.50	693.0	3.50	1400
<b>75</b>	3.75	742.5	3.75	1500
<b>80</b>	4.00	792.0	4.00	1600
<b>85</b>	4.25	841.5	4.25	1700
<b>90</b>	4.50	891.0	4.50	1800
<b>95</b>	4.75	940.5	4.75	1900
<b>100</b>	5.00	999.0	5.00	2000

Table 2. Demand dose, clinician bolus scroll ranges, milligrams

Concentration mcg/ml	Micrograms			
	Demand Dose		Clinician Bolus	
	increment	max.	increment	max.
<b>1</b>	0.05	9.9	0.05	20
<b>2</b>	0.10	19.8	0.10	40
<b>3</b>	0.15	29.7	0.15	60
<b>4</b>	0.20	39.6	0.20	80
<b>5</b>	0.25	49.5	0.25	100
<b>10</b>	0.50	99.0	0.50	200
<b>15</b>	0.75	148.5	0.75	300
<b>20</b>	1.00	198.0	1.00	400
<b>25</b>	1.25	247.5	1.25	500
<b>30</b>	1.50	297.0	1.50	600
<b>35</b>	1.75	346.5	1.75	700
<b>40</b>	2.00	396.0	2.00	800
<b>45</b>	2.25	445.5	2.25	900
<b>50</b>	2.50	495.0	2.50	1000
<b>55</b>	2.75	544.5	2.75	1100
<b>60</b>	3.00	594.0	3.00	1200
<b>65</b>	3.25	643.5	3.25	1300
<b>70</b>	3.50	693.0	3.50	1400
<b>75</b>	3.75	742.5	3.75	1500
<b>80</b>	4.00	792.0	4.00	1600
<b>85</b>	4.25	841.5	4.25	1700
<b>90</b>	4.50	891.0	4.50	1800
<b>95</b>	4.75	940.5	4.75	1900
<b>100</b>	5.00	990.0	5.00	2000
<b>200</b>	10.00	1980.0	10.00	4000
<b>300</b>	15.00	2970.0	15.00	6000
<b>400</b>	20.00	3960.0	20.00	8000
<b>500</b>	25.00	4950.0	25.00	10000

Table 3. Demand dose, clinician bolus scroll ranges, micrograms

Milliliters			
Demand Dose		Clinician Bolus	
increment	max.	increment	max.
0.05	9.9	0.05	20

Table 4. PCA delivery mode: Demand dose, clinician bolus scroll ranges, milliliters

---

## Specifications (Nominal)

### General Pump Specifications

---

#### Resolution

Medication Cassette reservoir or CADD® administration set, 0.050 ml/pump stroke nominal

CADD-Prizm® high volume administration set, 0.100 ml/pump stroke nominal

#### Size

4.4 cm x 10.4 cm x 14.1 cm (1.7 in. x 4.1 in. x 5.6 in.) excluding cassette or other accessories

#### Weight

568 g (20 oz.) including 9-volt battery and empty 100-ml Medication Cassette reservoir, excluding other accessories

#### Pump Alarms

Low battery power; depleted battery power; external power source low, faulty, depleted; pump stopped; pump fault; low reservoir volume; high delivery pressure; air in line; air detector faulty or detached (only with the use of the optional air detector); air detector port cover detached; delivery too slow; key stuck; cassette detached or unlocked; print failure, upstream occlusion (model 6101 only)

#### Bolus Volume at Occlusion Alarm Pressure

0.050 ml resolution sets/reservoirs:  
< 0.25 ml

0.100 ml resolution sets: < 2.0 ml

#### Power Sources

9-volt alkaline or lithium battery such as DURACELL® Alkaline MN 1604 or ULTRALIFE® Lithium U9VL; Power Pack reorder number 21-3801; AC adapter.

An internal battery powers the clock. When it is depleted, it cannot reliably maintain the clock time. This battery must be replaced by the manufacturer. The internal battery has an expected life of 5 years.

#### System\* Operating Temperature

+2°C to 40°C (36°F to 104°F)

#### System\* Storage Temperature

-20°C to 60°C (-4°F to 140°F)

#### Power Pack Charging Temperature

+10°C to 35°C (50°F to 95°F)

#### System\* Delivery Accuracy

± 6% (nominal)

#### High Pressure Alarm

18 ± 9 psi

#### Air Detector Alarm

Single bubble greater than 0.100 ml

### PCA Delivery Mode Specifications

---

#### Reservoir Volume

1 to 9999 or Not In Use; programmable in 1 ml increments, displayed in 0.1 ml increments  
Default: 1 ml

#### Units

Milliliters (ml), milligrams (mg), micrograms (mcg)  
Default: milligrams

#### Concentration

Mg/ml: 0.1, 0.2, 0.3, 0.4, 0.5, 1, 2, 3, 4, 5, 10, 15, ... 95, 100

Mcg/ml: 1, 2, 3, 4, 5, 10, 15, ...95, 100, 200, 300, 400, 500

Default: 1 mg/ml

#### Continuous Rate

0 to 30 ml/hr (or the mg or mcg equivalent)  
(See Table 1 for scroll ranges)  
Default: 0 mg/hr

#### Demand Dose

0 to 9.9 ml (or the mg or mcg equivalent)  
(See Tables 2, 3 and 4 for scroll ranges)  
Default: 0 mg/hr  
Delivery rate (Continuous Rate + Demand Dose): 125 ml/hr nominal

#### Demand Dose Lockout

5 minutes to 24 hours in the following increments:  
1 minute for values between 5 and 20 minutes  
5 minutes between 20 minutes and 24 hours  
Default: 5 minutes

#### Max Doses Per Hour

1 to 12 doses in 1 dose increments (will also be limited by the Demand Dose Lockout value)  
Default: 1

#### Demand Doses Given

0 to 999

#### Demand Dose Attempts

0 to 999

---

\*System is defined as a CADD-Prizm® pump with an attached Medication Cassette reservoir and CADD® extension set with integral anti-siphon valve, or an attached CADD® administration set with integral or add-on anti-siphon valve.

---

Given  
0 to 99999.99 in 0.01 unit increments

Clinician Bolus  
0.1 ml to 20.00 ml (or mg or mcg equivalent) (See Tables 2, 3 and 4 for scroll ranges) Delivery rate (Continuous Rate + Clinician Bolus): 125 ml/hr nominal

### Continuous Delivery Mode Specifications

Reservoir Volume  
1 to 9999 or Not In Use; programmable in 1 ml increments, displayed in 0.1 ml increments  
Default: 1 ml

Continuous Rate  
0.1 to 350 ml/hr in the following increments:  
0.1 for values between 0.1 and 100  
1 for values between 100 and 350  
Default: 0 ml/hr  
Use 9-volt battery for rates up to 250 ml/hr; use power pack or AC adapter for rates up to 350 ml/hr

Given  
0 to 99999.9 in 0.1 unit increments

### TPN Delivery Mode Specifications

Reservoir Volume  
10.0 to 9990 or Not In Use; programmable in 1 ml increments, displayed in 0.1 ml increments  
Default: 10 ml

Infusion Volume  
10 to 9990 ml in 10 ml increments:  
Default: 10 ml

Infusion Period  
0 hrs 10 min to 99 hrs 50 min in 10 minute increments:  
Default: 1 hr 0 min

Taper-Up Period  
0 hrs 0 min to 99 hrs 40 min in 10 minute increments:  
Default: 0 hrs 0 min

Taper-Down Period  
0 hrs 0 min to 99 hrs 40 min in 10 minute increments:  
Default: 0 hrs 0 min

Plateau Rate  
Calculated by pump; 10 to 350 ml/hr  
Use 9-volt battery for rates up to 250 ml/hr; use power pack or AC adapter for rates up to 350 ml/hr

KVO Rate  
Calculated by pump: 1/10 of Plateau Rate up to 5 ml/hr

Given  
0 to 99999 in 0.1 unit increments

### Intermittent Delivery Mode Specifications

Reservoir Volume  
1 to 9999 or Not In Use; programmable in 1 ml increments, displayed in 0.1 ml increments  
Default: 1 ml

Dose Volume  
0.1 to 1000 ml in the following increments:  
0.1 for values between 0.0 and 100  
1 for values between 100 and 1000  
Default: 0.0 ml

Dose Duration  
1 min to 24 hrs in the following increments:  
1 minute for values between 1 min and 10 min  
5 minutes for values above 10 min  
Default: 30 min  
Duration is limited by Dose Volume so that rate does not exceed 350 ml/hr. Use 9-volt battery for rates up to 250 ml/hr; use power pack or AC adapter for rates up to 350 ml/hr

Dose Cycle  
10 min to 96 hrs in 5 minute increments:  
Default: 4 hrs

KVO Rate  
0 to 10 ml/hr in 0.1 ml/hr increments  
Default: 0 ml/hr

Next Dose Start Time  
10 min to 96 hrs X min (where X equals a 10 minute increment in the 96th hour) or Immediate; programmable in 10 minute increments  
Default: Immediate

Given  
0 to 99999.9 in 0.1 unit increments

Time Remaining  
Dose and Cycle display in 1 minute increments

---

## Options Specifications

---

Immediate Taper-Down (TPN)  
0 to time remaining in the infusion period,  
in 10 minute increments (defaults to the  
currently programmed Taper-Down)

AutoLock  
Not In Use, LL1 or LL2

Time  
00:00 to 23:59

Air Detector  
Turned On or Turned Off

Event Log  
0 to 500 events

Extended History  
Up to 48 hours in 1 hour increments

## Biomed Toolbox Specifications

---

Micrograms  
On or Off

Extended History  
On or Off

Max Dose per Hour  
On or Off  
(PCA software rev. 6210G or higher only)

PM (Preventive Maintenance) Reminder  
1 to 24 months in 1 month increments,  
Not In Use

Custom Lock Level Code

Date Format  
US Standard (mm/dd/yy) or European  
Standard (dd/mm/yy)

Power Source Display  
Always display or only Low Battery

Upstream Sensor  
On or Off  
(Model 6101 only)

Air Detector Required  
Required or Not Required

## Compatible Reservoirs and Administration Sets

---

- 50-ml or 100-ml Medication Cassette reservoir, used with the CADD® extension set with anti-siphon valve.
- CADD® administration set with integral anti-siphon valve, with or without bag spike

(allows use of flexible plastic bag or sterile vial with injector)

- CADD® administration set with add on anti-siphon valve and bag spike (allows for gravity priming before attaching the add on anti-siphon valve)
- CADD-Prizm® high volume administration set with add on anti-siphon valve, with or without air-eliminating filter

## Remote Dose Cord

---

Deltec provides a Remote Dose Cord for the PCA delivery mode, which is an extension of the DOSE key. The push button switch is a Single Pole Double Throw (SPDT) which operates in the same manner as the DOSE key. When the Remote Dose Cord is attached to the pump, the patient may press either the Remote Dose button or the DOSE key to receive a Demand Dose. The clinician may also use either the Remote Dose button or the DOSE key to deliver a clinician bolus. For easy access, the Remote Dose cord may be fastened to the patient's clothing or bedsheet with the attached clip.

---

### NOTE:

To detach the Remote Dose cord from the pump, grasp the Remote Dose cord connector and pull back using a straight, steady motion. Do not twist or turn the connector, or use any instrument to remove it.

For additional specifications refer to the *Operator's Manual* provided with the product.

## 3 Batteries

---

### Battery Compatibility

#### Recommended Batteries

Nine-volt alkaline or lithium batteries are recommended for use in the CADD-Prizm<sup>®</sup> pump. Carbon-zinc, mercury, nickel-cadmium, or zinc-air 9-volt batteries should not be used.

#### Battery Life

The CADD-Prizm<sup>®</sup> pump has been designed to provide optimal battery life. The expected battery life in the CADD-Prizm<sup>®</sup> pump depends on the following factors:

- Programmed delivery rate
- Operating temperatures
- Frequency of display backlighting
- Frequency of printing
- Battery type and brand
- Battery age

---

### DURACELL<sup>®</sup> Alkaline Battery Life

The following tables may be used to predict typical alkaline battery life at different delivery rates when an alkaline battery is used in the CADD-Prizm<sup>®</sup> pump. As expected, battery life decreases as the delivery rate increases. These tables are based on laboratory tests using fresh DURACELL<sup>®</sup> alkaline batteries in CADD-Prizm<sup>®</sup> pumps while the pumps were operating at room temperature.

Actual battery life may be significantly shorter depending on the operating temperature and the storage conditions of the battery.

Battery life is shortened significantly at very low operating temperatures. For example, at 0°C (32°F), an alkaline battery will yield approximately 30% of its normal capacity.

Alkaline batteries do not need to be stored in a refrigerator. After four years of storage at 21°C

(70°F), an alkaline battery retains approximately 86% of its original capacity. Battery life will be shorter if the battery is stored above room temperature. An alkaline battery stored at 43°C (110°F) will be down to approximately 80% of its capacity within one year.

Recommended storage conditions are 10°C to 25°C (50°F to 77°F) with no more than 65% relative humidity noncondensing.

The following tables are based on laboratory tests conducted at room temperature using fresh DURACELL<sup>®</sup> alkaline batteries and a CADD<sup>®</sup> administration set. Actual battery life will vary depending on the brand of battery, battery shelf life and temperature conditions.

---

### ULTRALIFE<sup>®</sup> Lithium Battery Life

The following tables may be used to predict typical lithium battery life at different delivery rates when a lithium battery is used in the CADD-Prizm<sup>®</sup> pump. As expected, battery life decreases as the delivery rate increases. These tables are based on laboratory tests using fresh ULTRALIFE<sup>®</sup> lithium batteries in CADD-Prizm<sup>®</sup> pumps while the pumps were operating at room temperature.

Actual battery life may be significantly shorter depending on the operating temperature and the storage conditions of the battery. Lithium battery life is dependent upon the temperature and relative humidity of storage. Recommended storage conditions are less than 20°C (68°F) with a desiccant to ensure less than 10% relative humidity.

The following tables are based on laboratory tests conducted at room temperature using fresh ULTRALIFE<sup>®</sup> lithium batteries and a CADD<sup>®</sup> administration set. Actual battery life depends upon the brand of battery selected, the particular battery selected, battery shelf life, and temperature conditions. Deltec's testing indicates a large variability in battery life.

## Continuous and PCA Delivery Battery Life (Max Delivery Rate PCA Mode 30 ml/hr)

*Note: Results are without air detector.*

<b>Rate</b>	<b>Life</b>	<b>Volume</b>
0.4 ml/hr	120 hrs	48 ml
10 ml/hr	86 hrs	860 ml
30 ml/hr	37 hrs	1110 ml
50 ml/hr	26 hrs	1300 ml
100 ml/hr	13 hrs	1300 ml
200 ml/hr	14 hrs	2800 ml
350 ml/hr	7 hrs	2450 ml

Table 4. 9-volt Alkaline-type batteries used with the CADD-Prizm® pump.

<b>Rate</b>	<b>Life</b>	<b>Volume</b>
0.4 ml/hr	212 hrs	85 ml
10 ml/hr	161 hrs	1610 ml
30 ml/hr	79 hrs	2370 ml
50 ml/hr	60 hrs	3000 ml
100 ml/hr	30 hrs	3000 ml
200 ml/hr	32 hrs	6400 ml
350 ml/hr	17 hrs	5950 ml

Table 5. 9-volt Lithium-type batteries used with the CADD-Prizm® pump.

<b>Rate</b>	<b>Life</b>	<b>Volume</b>
100 ml/hr	64 hrs	6400 ml
200 ml/hr	67 hrs	13400 ml
350 ml/hr	39 hrs	13650 ml

Table 6. EPS System used with the CADD-Prizm® pump.

## Intermittent Delivery Battery Life

*Note: Results are without air detector.*

Volume	Duration	Cycle	KVO	Life	Volume
23.5	1:00 hr	5:00 hr	0.2 ml/hr	117 hrs	477 ml
61	1:00 hr	6:00 hr	0.2 ml/hr	71 hrs	733 ml
125	1:00 hr	6:00 hr	0.2 ml/hr	38 hrs	798 ml
200	1:00 hr	12:00 hr	0.2 ml/hr	84 hrs	1415 ml

Table 7. 9-volt Alkaline-type batteries used with the CADD-Prizm® pump.

Volume	Duration	Cycle	KVO	Life	Volume
23.5	1:00 hr	5:00 hr	0.2 ml/hr	301 hrs	1229 ml
61	1:00 hr	6:00 hr	0.2 ml/hr	183hrs	1891 ml
125	1:00 hr	6:00 hr	0.2 ml/hr	120 hrs	2520 ml
200	1:00 hr	12:00 hr	0.2 ml/hr	186 hrs	3134 ml

Table 8. 9-volt Lithium-type batteries used with the CADD-Prizm® pump.

## TPN Delivery Battery Life

*Note: Results are without air detector.*

Volume	Period	Taper Up	Taper Down	KVO	Life	Volume
2000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	12 hr	2333 ml
3000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	8 hr	2497 ml

Table 9. 9-volt Alkaline-type batteries used with the CADD-Prizm® pump.

Volume	Period	Taper Up	Taper Down	KVO	Life	Volume
2000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	32 hr	6333 ml
3000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	19 hr	5833 ml

Table 10. 9-volt Lithium-type batteries used with the CADD-Prizm® pump.

Volume	Period	Taper Up	Taper Down	KVO	Life	Volume
2000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	64 hr	12777 ml
3000	10:00 hr	1:00 hr	1:00 hr	5 ml/hr	45 hr	13498 ml

Table 11. EPS System used with the CADD-Prizm® pump.

## 4 Construction

The pump's housing is made of a special high impact plastic designed to reduce interference from electromagnetic fields and to dissipate electrostatic discharge. It is composed of two sections: the base and cover housing. The pump housing is sealed to ensure that the pump is water resistant. The battery compartment is not water resistant.

---

### NOTE:

The CADD-Prizm® ambulatory infusion pump is water resistant, but not waterproof.

The battery compartment is accessed through a removable door on the side of the base housing. Within the battery compartment is space for the battery and the two battery contacts.

The Medication Cassette reservoir or the administration set is attached to the bottom of the pump by inserting the two hooks on the cassette into the mating hinge pins on the pump. The pump and the reservoir or the administration set are then placed in an upright position on a firm, flat surface. The reservoir or the administration set can be latched in place by inserting a coin in the slot on the pump's latching button, pushing the button in, and turning the button one-quarter turn counterclockwise. The reservoir or the administration set is locked into place by inserting a key into the pump's lock and turning the lock one-quarter turn counterclockwise.

---

### NOTE:

The cassette lock must be unlocked before attempting to unlatch the disposable.

---

### NOTE:

The Medication Cassette reservoir and the administration set are intended for single use only.

The keyboard, located on the front housing, is composed of nine membrane switches and is sealed against moisture. All of the keys contain domes to provide a tactile feel when the key is pressed. The keyboard keys are sensed by the pump's microprocessor.

The custom Liquid Crystal Display (LCD), also located on the front housing, shows the pump status and programmed settings. The dot matrix display consists of 21 character columns with 4 rows of characters, and is selected by the pump's microprocessor according to status conditions and keyboard entries.

The microprocessor and other circuitry which control the pump are located on two printed circuit boards. The microprocessor board contains the Central Processing Unit (CPU) and its associated circuitry, motor driver circuitry, and other miscellaneous circuitry. The LCD board contains the Liquid Crystal Display with its associated circuitry, and the backlight module with its associated circuitry.

The pumping mechanism subassembly contains the motor, gear train, camshaft, valves, expulsor, sensing disk, infrared light source, infrared detector, occlusion sensor, cassette sensors, lock and latch. Via the motor driver circuitry, the pump's microprocessor controls motor rotation.

Two external port connectors are utilized for communication and external power input. One of these connectors, the data in/out jack, is used for attachment of the Remote Dose cord. This enables the patient to use either of two options to begin a Demand Dose when using the PCA delivery mode: (1) the Remote Dose button; or (2) the DOSE key.

This jack can also be connected to an external printer via the interface cable. With this feature, the patient or clinician can print various pump reports. The second port is for external power connection. This port, the power jack, can receive input from either an AC adapter or the External Power Source rechargeable power pack.

Connections between the printed circuit boards are designed for ease of manufacturing and serviceability. The keyboard is connected to the microprocessor board via a flex circuit tail. Flexible circuitry and discrete wires connect the pumping mechanism, motor, and sensors to the printed circuit boards.

## 5 Theory of Operation

---

### Keyboard Circuitry

The CADD-Prizm® pump is controlled by a microprocessor. The actions of the microprocessor are controlled by a program, which is contained in the memory.

Commands are issued to the microprocessor from the user via the nine keys on the keyboard and the Remote Dose cord. The keys on the keyboard feed individually into the Gate Array on the microprocessor board. A key closure applies a ground to the associated input of the Gate Array. Key debounce circuitry resident in the Gate Array provides a clean output signal to the microprocessor for the duration of the key closure. The microprocessor reads keyboard status by accessing special memory locations in the Gate Array.

The Remote Dose button consists of an SPDT switch with its own dedicated input to the microprocessor circuitry. The switch has a common input line and two output signal lines. The two signal lines are complementary such that one line is always logic high and the other is always low. When the Remote Dose button is pressed, both signal lines change to the alternate logic state. This redundancy prevents a single line failure from starting a dose delivery.

---

### Data Memory EEPROM

Many settings of the pump's delivery and record keeping parameters are stored by the microprocessor in an Electrically Erasable Programmable Read Only Memory (EEPROM). Data to and from the memory is presented serially. Whenever the microprocessor uses data from the EEPROM, the data is checked for validity.

---

### Battery Backed RAM

Additional settings of the pump's delivery and record keeping parameters are stored in a battery backed Random Access Memory (RAM). Battery backup is provided by two printed circuit board-mounted lithium batter-

ies. These batteries are designed to provide a minimum of five years of memory retention during normal pump usage. Whenever the microprocessor uses data from the RAM, the data is checked for validity.

---

### Time Base Circuitry

An accurate 3.6864 MHz timebase is provided by a quartz crystal. The 3.6864 MHz signal is connected to the microprocessor, where it is frequency-divided to access the program memory at a cycle rate of 921 kHz.

In addition, an accurate 32.768 kHz timebase is provided by a second quartz crystal. The 32.768 kHz signal is used for the real time clock.

---

### LCD Circuitry

The high-impedance, low-power, special drive signals for the liquid crystal display are provided by the LCD-drivers. Each alpha or numeric character on the LCD is formed by darkening combinations of dots. Commands to display dots are issued via data bus commands to the LCD-drivers by the microprocessor.

The LCD circuit also contains a power supply which provides bias voltage to the LCD panel. This voltage controls the relative brightness of the characters. Additional circuitry allows the microprocessor to disable the LCD when not in use in order to conserve battery power.

A two brightness level LCD backlight is provided to improve LCD viewing under low light conditions. When the microprocessor enables the LCD, it also enables the low brightness backlight. Low brightness is used to conserve battery power. If the AC adapter is connected, the microprocessor will enable the high brightness backlight since this does not consume power from the battery.

The backlight automatically shuts off when the LCD is turned off.

---

## LED Status Indicators

An amber and a green Light Emitting Diode (LED) are provided under the pump's front panel overlay to provide pump status to the user. Under software control, the LEDs can either flash at a low duty cycle or be on continuously. A flashing indicator typically indicates a normal mode of operation and a steady "on" indicator typically indicates a fault condition.

---

## Flash PROM Technology

Program memory for the pump is stored in Flash Programmable Read Only Memory (Flash PROM). This type of memory allows modification of the contents without physically removing the device from the circuit board. Under certain circumstances, the program can also be downloaded through the I/O port on the side of the pump. Several layers of redundancy in the programming system prevent accidental erasing or modification of the PROM.

---

## Gate Array Circuitry

The Gate Array contains circuitry which controls memory address decoding, keyboard debounce, Light Emitting Diode (LED) indicator status, LCD command buffering, Battery Backed RAM interface, and miscellaneous signal line buffering functions.

---

## Audible Alarm Circuitry

Audible alarm circuitry consists of a piezo electric disk and independent oscillator. The disk flexes or bends in resonance with the output of the oscillator. The piezo disk is mounted to the pump housing to enhance sound level. The oscillator which drives the piezo disk is capable of providing two driving frequencies. The low frequency is in the range of 700 to 1500 Hz and the high frequency is in the range of 1600 to 2500 Hz. The microprocessor controls the audible alarm via control lines from the Gate Array. When the microprocessor selects both the low and high frequency control lines, the audible alarm enters a warble mode where it oscillates between the low and

high frequency sound at a rate of 0.8 and 2 Hz. Low battery voltage detection and watchdog timer circuitry also have the ability to enable the audible alarm via the Gate Array.

---

## Watchdog Timer Circuit

Watchdog timer circuitry is provided to monitor the status of the microprocessor and disable the motor and enable the audible alarm if the microprocessor fails to function properly. The microprocessor must strobe the watchdog circuit at least once every second in order to prevent the watchdog from performing its reset function. The reset output from the watchdog circuit is a pulse output. This acts to "jump start" the microprocessor. This unique feature allows the microprocessor to test the watchdog circuit on every power-up. By setting a flag in memory and not strobing the watchdog, the microprocessor can force a watchdog time-out. After being reset, the microprocessor checks the status flag to see if this was a time-out test. If so, the microprocessor continues normal power-up activities. If the reset occurred when the microprocessor was not expecting it, the microprocessor traps the event, sounds the audible alarm and displays an error message on the LCD.

---

## Motor Driver/Motor Watchdog Circuit

Motor drive circuitry is composed of a series of power FET transistors, passive components, and two voltage comparators. Built into the motor drive circuitry is an RC timer which times how long the motor runs each time it is turned on. If the motor runs for more than an average of 4 seconds, the circuit will time out and disable the motor. A unique feature of this circuit is that control lines to and from the microprocessor circuit allow the microprocessor to perform a complete functional test of the motor drive circuit without running the motor. The microprocessor performs this test function every several minutes to assure its continued functionality. An input from the watchdog circuit prevents motor operation if the watchdog timer expires.

Rotation of the motor is sensed by the microprocessor via an infrared-sensitive photo

detector. An infrared light source is mounted so that its light beam illuminates the infrared detector. An opaque flag is mounted concentrically to the camshaft and rotates with it between the infrared light source and detector. When the flag interrupts the light beam, the output of the detector is sensed by the microprocessor via an input port bit. Power to the infrared LED light source is controlled by the motor driver circuit and is off when the motor is not running to conserve battery life.

In the microprocessor software, multiple checks are made on motion of the camshaft. When the motor is commanded to start, the infrared sensor must show that half a revolution has occurred within five seconds and that the motor has stopped when half a rotation was completed. In addition, no camshaft rotation can take place when the motor has not been commanded to run.

### Power Circuitry

Power for the pump is normally supplied by a 9-volt alkaline battery, 9-volt lithium battery, or AC adapter. These types of batteries have a fairly low internal resistance over their discharge range, which will keep power supply noise low. Other types of batteries, such as carbon-zinc, exhibit high internal resistance, especially near depletion. A voltage drop across the internal resistance occurs when current is drawn by the motor during pump activations. This current is demanded in short pulses when the motor is first turned on and generates large spikes in the battery voltage. This noise can cause the low battery detection circuit to shut down the pump.

The motor driver circuit power is taken directly from the battery, but the microprocessor and its associated circuitry requires closely regulated and filtered 5-volt power which is supplied from the micropower voltage regulator. This regulator will supply 5-volt power until its input voltage is approximately 5.3 volts. After that point, the output of the regulator will follow the input voltage down.

### Voltage Reference Circuit

A voltage reference circuit provides a constant DC voltage to the microprocessor Analog to Digital Converter (ADC). By reading this input and comparing the value to a predetermined range, the microprocessor can validate the accuracy of the 5-volt power supply. Variations in the 5-volt supply left undetected can result in inaccuracy in the low battery alarm set points and variations in other calculated values.

Voltage Trip Point*	CADD® Pump Status
>7.0V	No alarm
6.4–7.0V*	Transition to low battery condition; battery low message appears; 3 beeps every 5 min. <sup>†</sup>
6.0–6.6V*	Transition to depleted battery condition; battery depleted message appears; continuous alarm <sup>††</sup>
5.25–5.95V	Hardware reset occurs. Pump continues to indicate depleted battery condition.
* Voltage ranges are due to component tolerances. Actual trip values are guaranteed to be non-overlapping.	

Table 12. CADD-Prizm® pump low battery conditions.

† The pump emits 3 beeps every 5 minutes, and the message “9 Volt Battery Low” appears on the pump’s display, indicating that the battery power is low, but the pump is operable.

†† The pump emits a continuous, variable-tone alarm, and the message “9 Volt Battery Depleted” appears on the display, the battery power is too low to operate the pump, and pump operation has stopped.

### Pumping Mechanism

The pumping mechanism is linear peristaltic with two active valves. Pumping occurs when the expulsor presses on the reservoir pump tubing in sequence with the inlet and outlet valves. At rest, the outlet valve is pressing down fully on the tubing and the expulsor and inlet valve are retracted. (See Figure 7.)

When the microprocessor commands the mechanism to pump, the camshaft begins to rotate, thus controlling the following pump cycle:

1. The inlet valve closes.
2. In synchrony with the expulsor moving down to compress the tubing, the outlet valve opens, expelling 0.050 ml of fluid (or 0.100 ml of fluid with a CADD-Prizm® high volume administration set) toward the patient.
3. The outlet valve closes.
4. The inlet valve opens as the expulsor is retracted, causing fluid from the reservoir to again fill the pump tubing segment.
5. The camshaft rotation stops after half a revolution and the cycle is completed.

causes the pump mechanism to deliver 0.05 ml (or 0.1 ml with a CADD-Prizm® high volume administration set) fluid “pulses” timed according to the desired rate. At rates higher than 3 ml/hr, 2 pulses in succession will be given. Thus, to deliver 20 ml/hr, for example, the microprocessor solves these equations:

$$\begin{aligned} &\text{Mechanism activations per hr} \\ &= 20 \text{ ml per hr} / 0.1 \text{ ml per activation} \\ &= 20 / 0.1 \\ &= 200 \\ &\text{Time (seconds) between activations} \\ &= 3600 \text{ sec per hr} / \text{number of activations per hr} \\ &= 3600 / 200 \\ &= 18 \end{aligned}$$

### Pumping Characteristics

If the fluid path to the patient becomes blocked, the pump tubing will expand as pumping occurs. When there has been an amount of inflation corresponding to  $124 \pm 62 \text{ kPa}$  ( $1.24 \pm 0.62 \text{ bar}$ ,  $18 \pm 9 \text{ psi}$ ), the occlusion analog sensor trips, whereupon the microprocessor stops the pump mechanism and issues visual and audible alarms. Thus the maximum pressure which can be developed is 186 kPa ( $1.86 \text{ bar}$ ,  $27 \text{ psi}$ ).

To deliver the amount of drug specified by the parameter settings, the pump’s microprocessor

	Rate (ml/hr)	Volume Resolution (ml)
Cassette or Admin Set	0 - 3	0.050
	3.1 - 125	0.100
Hi Vol Admin Set	0 - 3	0.100
	3.1 - 350	0.200

The microprocessor uses its timer circuits to accurately time the 18 seconds (in this example) between mechanism activations. The timebase accuracy is ultimately determined by the 3.6864 MHz quartz crystal oscillator.

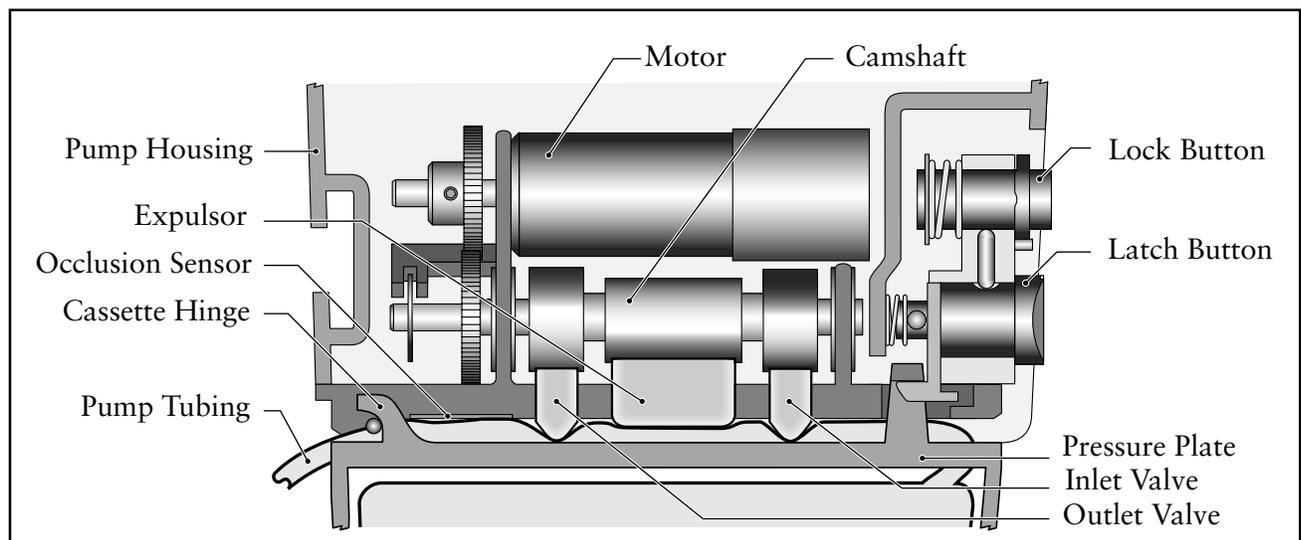


Figure 7. A simulated pumping mechanism in a CADD-Prizm® pump.

---

## Air Detector

The air detector is designed to detect air in the outlet tubing fluid path. The air detector is detachable if not needed. The CADD-Prizm® pump automatically detects the presence of the air detector and will automatically turn the sensor on when powered up in LL0.

When the optional air detector is installed, the Biomed Toolbox feature allows the air detector to be “required” or “not required.” When the air detector is not required, it can be “turned on” or “turned off” using the Options menu. When the air detector is required, the option for turning the air detector on or off will not be available. When the air detector is turned on, the pump will detect the presence of air in the outlet tubing fluid path. If the air detector settings are “not required” and “turned off,” it will default to “turned on” each time the pump powers up in Lock Level 0.

The air detector is compatible with all of the reservoirs and sets indicated for use with the CADD-Prizm® pump, and all pump accessories. It is powered directly from the CADD-Prizm® pump and no additional power is required.

### Specifications

The air detector will alarm when it senses a single air bubble greater than 100 microliters (0.1 milliliters.)

### Construction

The air detector housing is made of a special high impact plastic and has a metalized film coating on the inside surface to reduce interference from electromagnetic fields. The air detector is composed of a single base compartment with a detachable door. It is sealed against the pump housing to ensure the overall assembly is water resistant. The air detector is mounted to the pump housing with two screws, and electrically connected with a ten pin connector.

## Theory of Operation

The air detector consists of sensor electronics and two ultrasonic transducers positioned on opposite sides of the tubing. One transducer acts as an acoustic transmitter and the other as an acoustic receiver. Air detection occurs when air in the fluid path causes a reduction in the signal level to the receiver. When the signal is interrupted for a preset length of time, the sensing circuitry sends a signal to the microprocessor indicating air in the fluid path. To maximize the reliability of the system and to reduce false alarms, the transmitted signal is swept over a frequency range. This accommodates varying resonance frequencies of the transducer and reduces sensitivity to tubing tolerances and other mechanical variations.

---

## Upstream Occlusion Sensor

### Theory of Operation

The upstream occlusion sensor is a strain gauge device capable of detecting pressure changes in the disposable tubing set. This is accomplished by using a loading ball or sphere located on the bottom of the pump. This loading ball contacts the pump tubing when a tubing set is attached to the pump. Under normal operation, the pump tube pushes outward and applies a specified force on the sensor. When an upstream occlusion is present, the upstream tubing collapses pulling away from the sensor reducing the force on the sensor. It is this change of the force that indicates an upstream occlusion.

## 6 Safety Features and Fault Detection

### Hardware Safety Features

Key hardware safety features include a watchdog timer circuit, motor driver and motor watchdog circuits, cassette 'type' sensor circuit, latch/lock sensor circuit, and a voltage detector circuit. Each safety circuit performs a unique function to insure the overall safety of the device. (See Figure 8.)

### Watchdog Timer Circuit

The microprocessor must send an appropriate signal to the watchdog circuit at least once per second. If the microprocessor does not, the watchdog circuit will time out and shut down the pump controller.

Watchdog timer circuitry is provided to monitor the status of the microprocessor and disable the motor and enable the audible alarm if the

microprocessor fails to function properly. The microprocessor must strobe the watchdog circuit at least once every second in order to prevent the watchdog from performing its reset function. The reset output from the watchdog circuit is a pulse output. This acts to "jump start" the microprocessor. This unique feature allows the microprocessor to test the watchdog circuit on every power-up. By setting a flag in memory and not strobing the watchdog, the microprocessor can force a watchdog time-out. After being reset, the microprocessor checks the status flag to see if this was a time-out test. If so, the microprocessor continues normal power-up activities. If the reset occurred when the microprocessor was not expecting it, the microprocessor traps the event, sounds the audible alarm and displays an error message on the LCD.

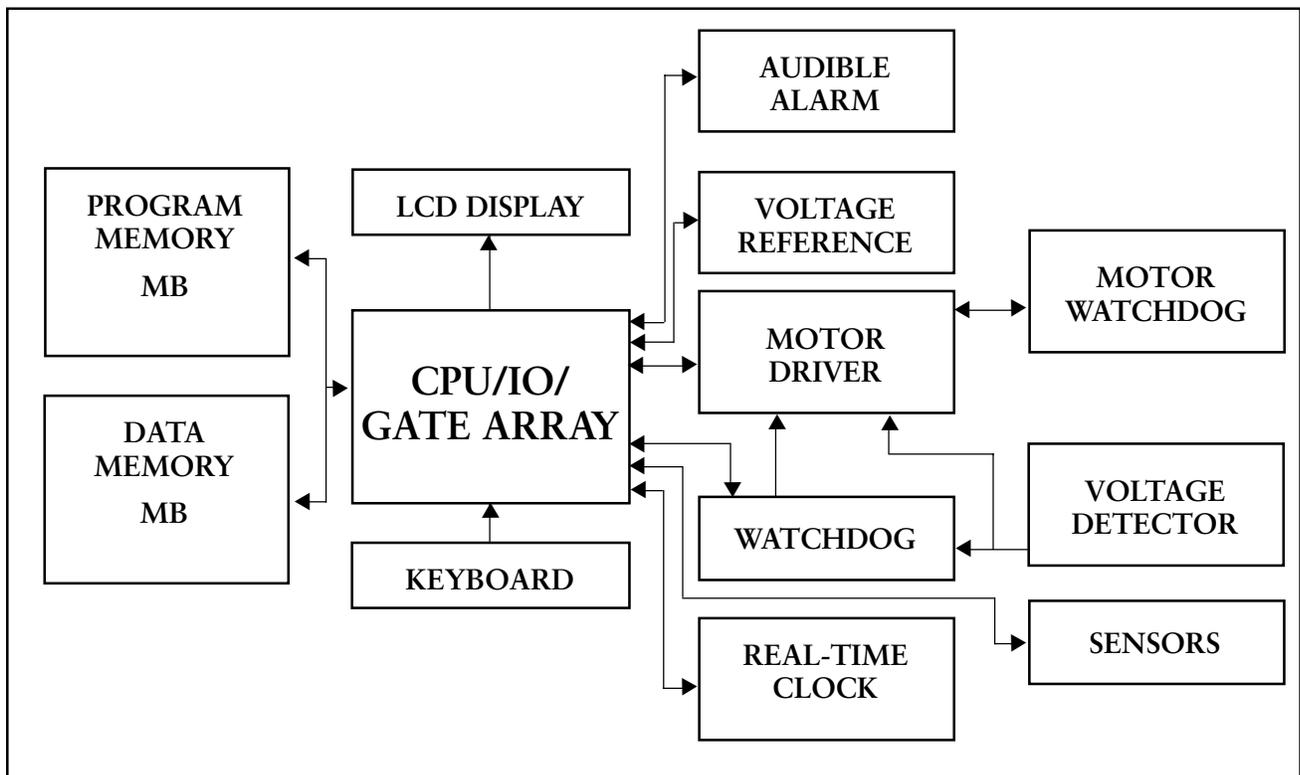


Figure 8. CADD-Prizm® pump hardware block diagram.

---

### Motor Driver/Motor Watchdog Circuit

Motor drive circuitry is composed of a series of power FET transistors, passive components, and two voltage comparators. Built into the motor drive circuitry is an RC timer which times how long the motor runs each time it is turned on. If the motor runs for more than an average of 4 seconds, the circuit will time out and disable the motor. A unique feature of this circuit is that control lines to and from the microprocessor circuit allow the microprocessor to perform a complete functional test of the motor drive circuit without running the motor. The microprocessor performs this test function every several minutes to assure its continued functionality. An input from the watchdog circuit prevents motor operation if the watchdog timer expires.

### Cassette 'Type' Sensor Circuit

The cassette 'Type' sensor system consists of three pins protruding from the button of the pump mechanism that interface to the attached administration set and associated circuitry. Each type of administration set designed to work with the CADD-Prizm<sup>®</sup> pump contains a unique 'code' programmed into the set via nubs molded into the plastic. When a set is latched to the pump, the nubs press against the pins in the pump mechanism in a pattern unique to that set type. Optical detectors and electronic circuitry on the circuit board encode this pattern and report the information to the microprocessor. This feature allows automatic rate selection dependent on the type of set attached. This system also acts as a safety feature to detect a damaged or detached set. If, during operation, the microprocessor detects all pins extended, the pump will enable audible and visual alarms and stop delivery. Redundancy in the pattern prevents single fault failures from causing over or under delivery of fluid. Additional circuitry allows these sensors to be turned on and off by the microprocessor to conserve battery power. Additionally, control of sensor power allows the microprocessor to test the sensor inputs in both the powered and unpowered states, thus allowing detection of sensor fault conditions. Care should be taken not to damage these sensor pins.

### Latch/Lock Sensor Circuit

Latch and Lock sensors allow the microprocessor to detect the positions of the latch and lock buttons. This prevents attempted fluid delivery when the set is not correctly latched to the pump. In addition, it allows the microprocessor to stop fluid delivery and enable audible and visual alarms if the set is unlatched during fluid delivery. Opposing infrared transmitters and receivers on both the latch and lock buttons allow the microprocessor to detect their open and closed positions. Additional circuitry allows these sensors to be turned on and off by the microprocessor to conserve battery power. Additionally, control of sensor power allows the microprocessor to test the sensor inputs in both the powered and unpowered states, thus allowing detection of sensor fault conditions.

### Voltage Detector Circuit

Low voltage detection is performed by part of the watchdog circuit and by the microprocessor via software. Three low voltage levels are detected. The first two levels are detected by software and the third by hardware. The first level to be reached is the Low Battery Warning threshold which occurs when the battery voltage decays to a nominal value of 6.8 volts. An Analog to Digital Converter (ADC) built into the microprocessor allows the microprocessor, via software, to monitor the battery voltage. At the Low Battery Warning threshold, the microprocessor enables a periodic series of beeps and displays a low battery warning message on the LCD. As the battery voltage reaches a nominal value of 6.3 volts, the software disables delivery, places a battery depleted message on the LCD, and enables a constant two-tone audible alarm. When the battery voltage decays to a nominal value of 5.6 volts, a hardware reset circuit is triggered which places the microprocessor in reset. This prevents ambiguous microprocessor operation when the battery voltage continues to decay. The hardware reset continues until the battery is completely discharged or until it is removed. Once the pump controller goes into low battery shutdown, only replacing the old battery with a fresh one will clear the condition.

---

## Software Safety Features

### Hardware-related Software Safety Features

#### Program Memory Check

At power up and at regular intervals thereafter, the program memory is tested by calculating a Cyclic Redundancy Code (CRC) on the program and then comparing it with the CRC stored with the program. If the stored and calculated CRCs do not match, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

#### RAM Memory Check

At power up, the random access memory is checked. A particular bit pattern is written to and read from each address in the RAM. If the read data is different from the written data, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

#### Motor Circuit Check

At power up and at regular intervals thereafter, the motor circuit is checked to ensure that no power is being applied to the motor unless the motor is actually on. If the software detects power being applied to the motor at any other time, it will sound a continuous two-tone audible alarm and will no longer attempt to deliver medication. During every pump activation, the software checks to see whether the motor completes one activation. If the motor fails to turn, or fails to complete a cycle, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

#### Keyboard Encoder Check

Every time the software receives data from the keyboard encoder, it is checked. If the data is not of the proper form, the software will turn on a continuous two-tone audible alarm and stop all drug delivery. The DOSE key has two independent signal lines to prevent single fault failures.

## Data Handling Software Safety Features

### Data Stored in RAM

Before use, data associated with delivery and stored in RAM is tested by calculating a CRC on the data and then comparing it with the CRC stored with the data. If the stored and calculated CRCs do not match, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

### Data Stored in EEPROM

Before use, data associated with delivery and stored in EEPROM is tested by calculating a CRC on the data and then comparing it with the CRC stored with the data. If the stored and calculated CRCs do not match, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

### Data Stored in NOVRAM

Before use, data associated with delivery and stored in NOVRAM is tested by calculating a CRC on the data and then comparing it with the CRC stored with the data. If the stored and calculated CRCs do not match, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

### Data Used in Calculations

Calculations on data used in some way to control the delivery of drug are performed redundantly. The two calculated values are then compared. If the two values do not match, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

### Timer Data Registers

The data stored in the timer control register is checked at regular intervals. If the data is not correct, the software will turn on a continuous two-tone audible alarm and stop all drug delivery.

# 7 Hardware and Software Fault Detection

## Overview

If the CADD-Prizm® pump displays an error code, a hardware or software fault has been detected by the microprocessor, and the pump should be returned for servicing.

When hardware or software faults are detected by the microprocessor, pump operation stops and a continuous, audible alarm will be activated as well as the amber warning LED. An error message will be displayed. On the next power up, the error code will again be displayed with the software level (see illustration below). If the error detected was a data fault, the pump will be in Lock Level 2, and all other programmed functions will have default values. (See the pump's *Operator's Manual* for specific defaults.)

Error code		Pump model number
C A D D - P R I Z M	lec XXXXX	61XX
sn XXXXXXXX	625 XXXXX	0254-01X XXXX-X
Terminal	Serial number	Software revision level

## Order of Error Code Events

1. There is a continuous two-tone audible alarm, a continuous amber indicator light, and the display will read



### NOTE:

- “XXXXXX” is a 5-digit code.
2. To silence the error code alarm, remove the battery.
3. At the next power-up, the last error code (lec) will be visible on the display. The microprocessor will also record an error code in the Event Log. The description “Error Detected” along with five digits will appear in the LCD in the Event Log. These five digits will remain in memory and will appear on the Event Log record until 500 data writes have occurred or until the mode has been changed. (See “Testing Procedures” starting on page 23 of this manual for detailed instructions regarding the power-up check.) Thus, there is always a record of the last internal fault detected by the microprocessor.

## 8 Cleaning and Inspection Procedures

### Inspection Recommendation

Deltec recommends annual functional inspection on the CADD-Prizm® pump. The following inspection and testing procedures should be performed annually to verify function and accuracy.

#### NOTE:

Persons performing the following tests and procedures should be familiar with the Deltec CADD-Prizm® pump. Please read the *Operator's Manual* supplied with the pump before proceeding.

#### WARNING:

CADD® pumps are sealed units. A broken or damaged seal will, therefore, be considered conclusive evidence that the pump has been misused and/or altered, which voids any and all warranties. All service and repair of CADD® pumps must be performed by Deltec or its authorized agents.

### Cleaning

#### CAUTION:

- Do not immerse the pump in cleaning fluid or water. Do not allow solution to soak into the pump, accumulate on the keypad, or enter the battery compartment.
- Do not allow solution to enter the data in-out jack or the power jack. Make sure the jack covers are closed before cleaning.
- Do not expose the open air detector port area of the pump or the connector opening on the air detector to foreign material, moisture or cleaning fluids. If an air detector is not installed on the pump, make sure the air detector port cover is securely attached before cleaning.
- Do not clean the pump with acetone, other plastic solvents, or abrasive cleaners.

Use any of the following solutions to clean the pump and accessories:

- Soap solution
  - Benzalkonium chloride concentrate (0.13%)
  - Glutaral concentrate, USP (2%)
  - 10 percent solution of household bleach (one part household bleach to nine parts water)
  - Alcohol, USP (93%)
  - Isopropyl Alcohol, USP (99%)
  - PDI – Super Sani-Cloth®
  - Mada Medical – MadaCide
1. Dampen a soft, lint-free cloth with cleaning solution. Apply the solution to exterior surface of the pump or accessory. *Do not allow the solution to soak into the pump or accessory.*
  2. Wipe the entire surface dry with another soft, lint-free cloth. Allow the pump to dry completely before use.

### Visual Inspection

- Visually inspect the pump for any damage to the LCD, occlusion sensor seals, valves and expulsor, reservoir hinge area, latch, lock, cassette sensors (3), keyboard, indicator lights, power jack, data in/out jack, air detector port cover or air detector, and housing. If any damage is noted, the pump should be returned for service.
- Check the battery door for proper operation. It should not be broken or damaged. The mating tabs on the pump housing should not be broken or damaged.
- Examine the battery compartment for damage. If the battery contacts appear corroded, clean them with a cotton swab and isopropyl alcohol. If the battery contacts appear to be bent or pushed in, straightening may be possible with a small screwdriver or other suitable tool. Care must be taken so as not to damage the pump housing or to incur further damage to the contacts.

---

## Mechanical Inspection

- Press each key on the keyboard. Each key should have a distinctive dome feeling. The keys should not feel flat.
- Attach the battery door. The battery door should fit snugly in place when it is closed on the pump.
- Attach either a 50- or 100-ml Medication Cassette reservoir or a CADD® administration set to the pump. Check for smooth operation and a definite “feel” when the latch pulls the reservoir or administration set firmly against the bottom of the pump. The mark on the latch should be aligned with the solid dot.
- Lock the device by inserting a key into the lock and turn counterclockwise until the mark lines up with the solid dot.
- Gently twist and pull on the cassette to make sure it is firmly attached.

---

### NOTE:

In the PCA delivery mode, the cassette must be locked in order to start the pump.

# 9 Testing Procedures

## Functional Testing

The Power-up Check, Latch/Lock Check and Cassette Sensor Check using a Medication Cassette reservoir or CADD® administration set can be performed with the pump in any of the four delivery modes.

The Cassette Sensor Check using the CADD-Prizm® high volume administration set can be performed in any delivery mode *except* the PCA delivery mode.

### Power-up Check

- Insert a battery in the pump and observe the LCD during power up.

Error code		Pump model number
C A D D - P R I Z M		61XX
lec XXXXX		
sn XXXXXXXX		0254-01X
625 XXXXX		XXXX-X
Terminal	Serial number	Software revision level

The number values may vary depending on the current mode and software revision.

- If “lec XXXXX” (last error code with 5 digit number) appears on the display prior to the pump reviewing the current program settings, the pump has experienced an electrical or mechanical fault and should be returned for service. If no error message is immediately shown, the pump has powered up normally. The pump should sequentially display all of the programmed values. The words “Self Test Complete” should appear, then the text “Power Up Successful” with six audible beeps. Continue with the Latch/Lock check.

### Latch/Lock Check

- Attach a 50- or 100-ml Medication Cassette reservoir or a CADD® administration set to the pump. The mark on the latch should be aligned with the solid dot. The display should show that the reservoir or administration set is latched. See Table 13.

<b>Reservoir Latched</b>	The Medication Cassette reservoir has been latched onto the pump.
<b>Admin set Latched</b>	The CADD® administration set has been latched onto the pump.
<b>High Volume Admin Set Latched</b> (excluding PCA delivery mode)	The CADD-Prizm® high volume administration set has been latched onto the pump.

Table 13.

- Lock the device by inserting a key into the lock and turning counterclockwise until the mark lines up with the solid dot. The display should show “Cassette Locked.”
- Unlock the device by inserting a key into the lock and turning clockwise until the mark lines up with the open dot. The display should show “Cassette Unlocked.”
- Unlatch the reservoir by inserting a coin into the latch slot and turning clockwise until the mark lines up with the open dot. The display should show “Cassette Unlatched / Close Clamp to Prevent Free Flow.”

### Cassette Sensor Check

- Attach a 50- or 100-ml Medication Cassette reservoir to the pump. Latch the cassette to the pump. The display should show “Reservoir latched.” NOTE: The message displayed depends on the type of reservoir or administration set attached. See Table 13.
- Lock the device by inserting a key into the lock and turning counterclockwise until the mark lines up with the solid dot. The display should show “Cassette Locked.”
- Unlock the cassette. The display should show “Cassette Unlocked.” Unlatch the cassette. The display should show “Cassette Unlatched / Close Clamp To Prevent Free Flow.”

- Remove the 50- or 100-ml reservoir and attach a CADD® administration set to the pump. Latch the cassette to the pump. The display should show “Admin Set Latched.”
- Lock the device by inserting a key into the lock and turning counterclockwise until the mark lines up with the solid dot. The display should show “Cassette Locked.”
- Unlock the cassette. The display should show “Cassette Unlocked.” Unlatch the cassette. The display should show “Cassette Unlatched/ Close Clamp To Prevent Free Flow.”
- Remove the administration set and attach a CADD-Prizm® high volume administration set to the pump (excludes PCA delivery mode). Latch the cassette to the pump. The display should show “High Volume Admin Set Latched.”
- Lock the device by inserting a key into the lock and turning counterclockwise until the mark lines up with the solid dot. The display should show “Cassette Locked.”
- Unlock the cassette. The display should show “Cassette Unlocked.” Unlatch the cassette. The display should show “Cassette Unlatched/ Close Clamp To Prevent Free Flow.”

- Program the pump to the following parameters:

Reservoir Volume:	2.0 ml
Units:	Milligrams
Concentration:	1.0 mg/ml
Continuous Rate:	30.0 mg/hr
Demand Dose:	0.0 mg
Milligrams Given:	0.0 mg (press the ENTER key to clear)

- Press the NEXT key until reservoir volume is displayed on the LCD. Press the Y or N key until 2.0 ml is displayed. Then press the ENTER key. Select Milligrams for units and press the ENTER key. Select the Concentration of 1.0 mg and press the ENTER key. Select the Continuous Rate of 30.0 mg/hr, then press the ENTER key. Select the Demand Dose of 0.0 mg, then press the ENTER key. Clear the Milligrams Given register by pressing the ENTER key.

### Motor and Gear Train Check

- Attach either a 50- or 100-ml Medication Cassette reservoir or CADD® administration set to the pump. Latch and lock the cassette.
- Select “Prime” from the Options menu by pressing the OPTIONS key, then the ENTER key. Now press and continue to hold the Y key. The pump should begin to prime. While priming the pump, listen to the motor for excessive noise or grinding sounds. Count the number of pump activations. The pump should prime ten double activations and then stop. The display should show “Continue Priming? Press Y or N.” Press the N key. Then press the NEXT key twice until the Reservoir Volume screen appears. The reservoir volume should show 1.0 ml.

### Reservoir Volume is Zero Alarm Check

- Press the OPTIONS key and select the Prime function by pressing the ENTER key. Repeat priming by pressing and holding the Y key. The pump should prime ten double activations and then stop. The pump will alarm and display “Reservoir Volume is Zero.” Press the NEXT key.
- Reprogram the reservoir volume to 1.0 ml. Press the NEXT key until Reservoir Volume is displayed on the LCD. Press the Y or N key until 1.0 ml is displayed. Then press the ENTER key.

### NOTE:

The basic electrical and mechanical functions of the CADD-Prizm® pump are the same for each of the four delivery modes. Regardless of the delivery mode, the delivery accuracy and timing circuitry remains unchanged. Although functional testing of the CADD-Prizm® pump can be performed in any mode, the remaining tests are intended to be performed in the PCA delivery mode. Additional tests may be performed with the pump in a different delivery mode if desired.

The following three checks (LCD, motor and gear train, and Reservoir Volume is Zero alarm) should be performed in the sequence shown.

### LCD Check

- Remove and reinsert the battery. After a few seconds, the LCD will display all off pixels (dots) followed by all on pixels. Examine the LCD for missing dark or light pixels.

---

**NOTE:**

The remaining testing procedures should be performed using a 50- or 100-ml Medication Cassette reservoir containing fluid and a primed extension set with anti-siphon valve or a primed CADD® administration set with anti-siphon valve.

**Starting/Stopping the Pump**

- Check the STOP/START key by pressing it. “Start the Pump?” should be displayed. Press the Y key. The display should show “Starting Pump” followed by a review of the programmed parameters. The main screen should appear with “RUNNING” in the display, and the green LED indicator light should blink every 3 seconds.
- To stop the device, press the STOP/START key. When the message “Stop the Pump?” appears, press Y. STOPPED appears in the display and the amber LED indicator light blinks.

**Activation Timing Check**

- Check the activation timing by programming the pump with the following values:

Reservoir Volume:	1.0 ml
Units:	Milligrams
Concentration:	1.0 mg/ml
Continuous Rate:	30.0 mg/hr
Demand Dose:	0.0 mg
Milligrams Given:	0.0 mg (press the ENTER key to clear)
- Press the STOP/START key. Press the Y key. “Starting Pump” should appear on the display. The pump should sequentially display all of the programmed values. Start a timer at the first motor activation.
- Count the activations. One activation should occur every twelve seconds. Approximately one minute fifty seconds (1:50) and ten activations later, the RES VOL alarm should occur. The display should show “Reservoir Volume is Zero” with a Milligrams Given of 1.0 mg.

**DOSE Key Check**

- Check the DOSE key operation by programming the pump with the following values:

Reservoir Volume:	10.0 ml
Units:	Milligrams
Concentration:	1.0 mg/ml
Continuous Rate:	0.0 mg/hr
Demand Dose:	1.0 mg
Demand Dose Lockout:	0 hrs 5 min
Max Doses Per Hour:	12
Dose Counters:	0/0 (Press the ENTER key to clear)
Milligrams Given:	0.0 mg (Press the ENTER key to clear)
- Press the STOP/START key. Press the Y key. The pump should sequentially display all of the programmed values.
- After RUNNING appears on the display, press the DOSE key and note the time. The pump should beep twice and begin to deliver. Count the number of pump activations. The pump should make ten double activations. After ten double activations, the display should show a reservoir volume of 9.0 ml. Press the DOSE key two more times within the next 5 minutes. The pump should not deliver and the message “Dose Not Delivered Dose Locked Out” should be displayed.

**Remote Dose Cord Check (if applicable)**

- Wait 5 minutes after the dose given above; then, instead of pressing the DOSE key, press the button on the Remote Dose cord. The pump should make ten double activations. After ten double activations, the display should show a reservoir volume of 8.0 ml. Press the DOSE key two more times within the next 5 minutes. The pump should not deliver and the message “Dose Not Delivered Dose Locked Out” should be displayed.

**Doses Given and Doses Attempted Check**

- Stop the pump by pressing the STOP/START key, then the Y key. Use the NEXT key to advance to the Dose Counters screen. The display should show 2/6, if the DOSE key and Remote Dose cord were tested. If only the DOSE key was tested, the display will show 1/3. (If the above steps have not been followed exactly, different values may appear.)

- Press the ENTER key. The display should now show 0/0.

### MG GIVEN Mode Check

- Press the NEXT key to advance to the Milligrams Given screen. The display should now show 2.0 mg, if the DOSE key and Remote Dose cord were tested. If only the DOSE key was tested, the display will show 1.0. (If the above steps have not been followed exactly, different values may appear.)
- Press the ENTER key. The display should now show 0.0 mg.

---

### Air Detector Test (if applicable)

This test will verify the function of the optional air detector. To perform this test, the CADD-Prizm<sup>®</sup> pump must have an air detector installed and the air detector must be turned on. The previous program from the DOSE key check (page 26) can be used to perform this test.

- Attach an empty Medication Cassette reservoir or CADD<sup>®</sup> administration set to the pump.
- Latch and lock the set to the pump.
- Open the air detector door and thread the tubing through the groove.
- Close the door making sure the tubing does not get pinched or kinked.
- Start the pump.
- The pump should respond with a continuous two-tone alarm and the display should read:

Air in line detected  
Pump will not run  
Next to silence

- Press NEXT to silence the alarm, and remove the Medication Cassette reservoir or CADD<sup>®</sup> administration set.
- Now attach a Medication Cassette reservoir containing fluid and a primed extension set with anti-siphon valve, or a primed CADD<sup>®</sup> administration set with anti-siphon valve to the pump. Make certain there is no air in the fluid path.

- Latch and lock the set to the pump.
- Open the air detector door and thread the tubing through the groove.
- Close the door making sure the tubing does not get punched or kinked.
- Start the pump.
- Deliver a demand dose. (NOTE: five minutes must have passed since the delivery of the last demand dose.)
- The pump should deliver the dose without an air detection alarm.

---

### Occlusion Accuracy Tests

This manual provides two testing options for the Occlusion Pressure Range Test and the Accuracy Test. Only one option needs to be performed. It is not necessary to perform both Option 1 and Option 2.

#### Downstream Occlusion Pressure Range Test (Option 1)

##### Description

Pressure is generated by activating the pumping mechanism with an attached filled, clamped Medication Cassette reservoir. The pump is started and a Demand Dose is given until the high pressure alarm sounds.

##### Equipment needed

50- or 100-ml Medication Cassette reservoir containing water.

##### Procedure

1. Insert a battery and wait for the pump to power up.
2. Attach a Medication Cassette reservoir containing water to the pump. Latch and lock the cassette.
3. Prime the Medication Cassette reservoir tubing. The tubing should be filled with fluid to the end of the luer lock connector. The system *must* be free from air bubbles for this test.
4. Close the slide clamp on the distal end of the tubing near the female luer of the Medication Cassette reservoir.

- Program the pump to the following parameters:

Reservoir Volume: 10.0 ml  
 Units: Milligrams  
 Concentration: 1.0 mg/ml  
 Continuous Rate: 0.0 mg/hr  
 Demand Dose: 1.0 mg  
 Demand Dose Lockout: 0 hrs 5 min  
 Max Doses Per Hour: 12  
 Dose Counters: 0/0 (Press the ENTER key to clear)  
 Milligrams Given: 0.0 mg (Press the ENTER key to clear)

- Start the pump. When the pump is running, activate a Demand Dose, noting when the high pressure alarm is activated.
- The pump should alarm when the pump delivers between 1 and 2 activations.

#### **CAUTION:**

At the completion of the test, the pressure must be reduced to zero before detaching the cassette from the pump; otherwise, the cassette may rupture. Safety glasses should be worn while conducting or observing this test.

### **Downstream Occlusion Pressure Range Test (Option 2)**

#### **Description**

An adjustable metered pressure source is connected to the Medication Cassette reservoir tubing. The pressure is slowly increased until the high pressure alarm sounds.

#### **Equipment needed**

- Pressure gauge, 30 psi  $\pm$  1 psi.
- Pressure vessel, partially filled with water.
- Pressure regulator, 30 psi.
- 50 or 100 ml Medication Cassette reservoir containing water.

#### **Procedure**

- Insert a battery and wait for the pump to power up.
- Attach a Medication Cassette reservoir to the pump. Latch and lock the cassette.

#### **NOTE:**

The pressure from the source must be zero when the cassette is attached.

- Assemble the apparatus as shown in Figure 9.

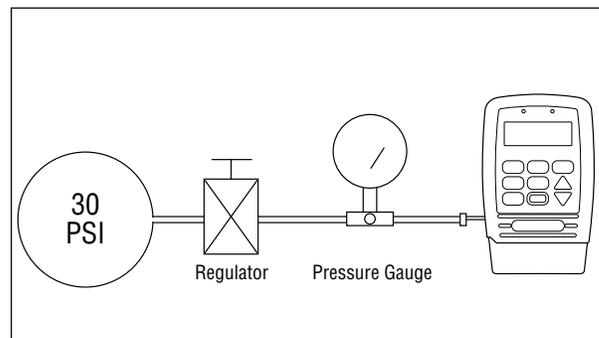


Figure 9. Occlusion test set-up.

- Connect the Medication Cassette™ reservoir outlet tube to the metered pressure source.

#### **NOTE:**

Do not use a CADD® extension set with anti-siphon valve.

- Start the pump and run at 30 ml/hr.
- Slowly increase the back pressure, noting when the high pressure alarm is activated. NOTE: The pressure may be increased rapidly to 8 psi, after which the pressure should be increased at 3 psi/min or less until the alarm sounds.
- The high pressure alarm should sound between 9 and 27 psi (18 $\pm$ 9 psi).

### **Upstream Occlusion Sensor Test (Model 6101 pumps only)**

#### **Description**

The tubing between the fluid reservoir and the pump is occluded while the pump is running until the occlusion alarm sounds.

#### **Equipment needed**

- CADD® administration set with anti-siphon valve.
- Tubing clamp (slide clamp or hemostat)

---

## Procedure

1. Spike an appropriate standard I.V. bag.
2. Prime the entire fluid path.
3. Program the pump to deliver a continuous rate of 20 ml/hr.
4. Start the pump.
5. Clamp the tubing halfway between the fluid reservoir and the pump.
6. The pump should alarm within three activations after clamping the tubing.

**Note:** Make sure the upstream occlusion sensor is turned on in the Biomed Toolbox.

---

## Accuracy Testing

Accuracy testing may be performed in any delivery mode. This manual shows an example of delivery accuracy testing with the pump in the PCA delivery mode.

### Gravimetric Accuracy Testing (Option 1)

#### Description

A Medication Cassette reservoir is partially filled with water and weighed. The reservoir is then attached to the pump and the pump is set to deliver a certain amount of water. The reservoir is then removed and weighed again. The amount of water delivered is compared to the amount that the pump should have delivered.

Nominal system accuracy is given in the technical specifications section for the pump. That is, under the test conditions described below, the accuracy of the pump and reservoir will be nominal with a 90% confidence level. The nominal test conditions are as follows: degassed water at  $25 \pm 5^\circ\text{C}$  without back pressure.

#### Equipment needed

- 50- or 100-ml Medication Cassette reservoir.
- 50- or 60-ml syringe.
- CADD® extension set with anti-siphon valve
- A balance accurate to 0.1 g.
- 40 ml of room temperature water.

## Procedure

1. Fill the 50- or 60-ml syringe with 40 ml of water. Transfer the water into a Medication Cassette reservoir.
2. Remove any air from the Medication Cassette reservoir by aspirating the air with the syringe. Attach the CADD® extension set with anti-siphon valve. Prime the tubing so it is filled with fluid to the end of the extension set luer lock connector.
3. Secure the clamp as close to the extension set luer lock connector as possible. This should assure a minimum water loss from the tubing when the syringe is removed.
4. Weigh the entire reservoir/extension set assembly and record the weight. This is the pre-delivery weight. (This weight includes the empty Medication Cassette reservoir, extension set, and weight of the water.)
5. Attach the Medication Cassette reservoir to the pump. Program the reservoir volume to 20 ml. Now press the ENTER key. This value is the intended delivery volume. (One ml of water at  $20^\circ\text{C}$  weighs 1 gram.) Open the clamp.
6. With the pump in Lock Level 0, program a continuous rate of 0 ml/hr and a dose of 1.0 ml (but do not deliver a Demand Dose). Start the pump and deliver a clinician bolus of 20 ml.
7. Again, secure the clamp as close as possible to the end of the extension set luer lock connector. Remove the Medication Cassette reservoir from the pump and weigh the entire reservoir/extension set assembly. This is the **postdelivery weight**.
8. Calculate the difference in weight between the pre-delivery weight and the postdelivery weight. This is the **weight of the amount delivered**.
9. Find the difference between the volume of the amount delivered and the intended delivery volume. This is the **inaccuracy volume**.
10. Divide the inaccuracy volume by the intended delivery volume and multiply by 100. This is the **accuracy error percentage** (See Table 14).

Pre-Delivery Weight	Post Delivery Weight	Weight of Amount Delivered	Intended Delivery Volume	Inaccuracy Volume	Accuracy Error	Accuracy Error Percentage
61.1 g	41.6 g	19.5 g = 19.5 ml	20 ml	-0.5 ml	$-0.5 \text{ ml} \div 20.0 \text{ ml} = -0.025$	$-0.025 \times 100 = -2.5\%$

Table 14. Gravimetric percentage calculation

11.If the accuracy error percentage is greater than  $\pm 6\%$ , repeat the test with a new reservoir. If the pump fails a second time, call Deltec's Customer Service Department.

#### EXAMPLE:

Predelivery Weight:	61.1 g
Postdelivery Weight:	- 41.6 g
<b>Weight of Amount Delivered:</b>	<b>19.5 g</b> <b>= 19.5 ml</b>
Volume of Amount Delivered:	19.5 ml
Intended Delivery Volume:	- 20.0 ml
<b>Inaccuracy Volume:</b>	<b>- 0.5 ml</b>
Inaccuracy Volume:	- 0.5 ml
Intended Delivery Volume:	$\div 20.0 \text{ ml}$
<b>Accuracy Error:</b>	<b>-0.025 ml</b>
Accuracy Error:	-0.025
	$\times 100.00$
<b>Accuracy Error Percentage:</b>	<b>-2.5%</b>

### Volumetric Accuracy Testing (Option 2)

#### Description:

A predetermined amount of water is delivered into a collection device such as a burette or graduated cylinder. The amount of water delivered is compared to the amount that the pump should have delivered.

Nominal system accuracy is given in the technical specifications section for the pump. That is, under the test conditions described below, the accuracy of the pump and reservoir will be nominal with a 90% confidence level. The nominal test conditions are as follows: degassed water at  $25 \pm 5^\circ\text{C}$  without back pressure.

#### Equipment needed:

- 50- or 100-ml Medication Cassette reservoir
- 50- or 60-ml syringe
- CADD® extension set with anti-siphon valve
- A fluid collection device such as a burette or a class A 25 ml capacity graduated cylinder
- 40 ml of room temperature water

#### Procedure:

1. Fill the 50- or 60-ml syringe with 40 ml of water. Transfer the water into a Medication Cassette reservoir.
2. Remove any air from the Medication Cassette reservoir by aspirating the air with the syringe. Attach the CADD® extension set with anti-siphon valve. Prime the tubing so it is filled with fluid to the end of the extension set luer lock connector.
3. Attach the end of the extension set to the fluid collection device.
4. Attach the Medication Cassette reservoir to the pump. Program the reservoir volume to 20 ml. This is the **intended delivery volume**. Open all clamps.
5. Program a continuous rate of 0.0 ml/hr and a Demand Dose of 1.0 ml (but do not deliver a Demand Dose). Start the pump and deliver a clinician bolus of 20 ml.
6. When delivery is complete, record the volume of fluid delivered. This is the **actual delivery**.
7. Find the difference between the volume of the amount delivered and the intended

---

delivery volume. This is the **inaccuracy volume**.

8. Divide the inaccuracy volume by the intended delivery volume and multiply by 100. This is the **accuracy error percentage**. (See Table 15.)

9. If the accuracy error percentage is greater than  $\pm 6\%$ , repeat the test with a new reservoir. If the pump fails a second time, call Deltec's Customer Service Department.

Intended Delivery Volume	Actual Delivery Volume	Inaccuracy Volume	Accuracy Error	Accuracy Error Percentage
20 ml	19.5 ml	-0.5 ml	$-0.5/20.0 \text{ ml} = -0.025$	$-0.025 \times 100 = \pm 2.5\%$

Table 15. Volumetric percentage calculation

## CADD-Prizm® Pump Cleaning and Functional Testing Checklist

The following checklist is provided as a guide only to assist in establishing documentation of cleaning and functional testing for the CADD-Prizm® pump. If service is provided, fill out this sheet and return it with the device.

Serial # \_\_\_\_\_ Reference Number \_\_\_\_\_ Date \_\_\_\_\_

(Refer to the Technical Manual procedures.)

I. Cleaning Completed  Yes  No

II. Visual Inspection

- |   |   |   |
|---|---|---|
| <input type="checkbox"/> LCD                    | <input type="checkbox"/> Lock                 | <input type="checkbox"/> Data In/Out Jack           |
| <input type="checkbox"/> Occlusion Sensor Seals | <input type="checkbox"/> Cassette Sensors (3) | <input type="checkbox"/> Air Detector or Port Cover |
| <input type="checkbox"/> Valves and Expulsors   | <input type="checkbox"/> Keyboard             | <input type="checkbox"/> Pump Housing               |
| <input type="checkbox"/> Reservoir Hinge Area   | <input type="checkbox"/> Indicator Light      | <input type="checkbox"/> Battery Door               |
| <input type="checkbox"/> Latch                  | <input type="checkbox"/> Power Jack           | <input type="checkbox"/> Battery Compartment        |

III. Mechanical Inspection

- |                                       |   |
|---------------------------------------|---|
| <input type="checkbox"/> Keyboard     | <input type="checkbox"/> Cassette Latch |
| <input type="checkbox"/> Battery Door | <input type="checkbox"/> Cassette Lock  |

IV. Functional Inspection

- |   |  |   |
|---|--|---|
| <input type="checkbox"/> Power-up           | <input type="checkbox"/> RES VOL Alarm     | <input type="checkbox"/> Dose Given / Attempted |
| <input type="checkbox"/> Latch / Lock       | <input type="checkbox"/> Stop / Start      | <input type="checkbox"/> MG Given               |
| <input type="checkbox"/> Cassette Sensor    | <input type="checkbox"/> Activation Timing | <input type="checkbox"/> Air Detector           |
| <input type="checkbox"/> LCD                | <input type="checkbox"/> Dose Key          | <input type="checkbox"/> Printer Tests          |
| <input type="checkbox"/> Motor / Gear Train | <input type="checkbox"/> Remote Dose Cord  |   |

V. Occlusion Tests (Only need to perform 1 or 2; not both)

Downstream Option 1: Activations Before Alarm \_\_\_\_\_

Downstream Option 2: High Pressure Alarm At \_\_\_\_\_ psi

Upstream Occlusion Sensor Test

Pass \_\_\_\_\_ Fail \_\_\_\_\_

VI. Accuracy Testing (Only need to perform 1 or 2; not both)

Volumetric Accuracy Test

Intended Delivery Volume	Actual Delivery Volume	Inaccuracy Volume	Accuracy Error	Accuracy Error Percentage
ml	ml	ml		%

Gravimetric Accuracy Test

Pre-Delivery Weights	Post-Delivery Weight	Amount Delivered	Intended Delivery Volume	Inaccuracy Volume	Accuracy Error	Accuracy Error
g	g	ml	ml	ml		%

**Customer & Clinical Services:**

**1-800-426-2448** U.S. & Canada

**www.deltec.com**

## **Deltec**

**Deltec, Inc.**

St Paul MN 55112 USA

CADD, CADD-Prizm and Medication Cassette reservoir design are Deltec, Inc. trademarks.

DURACELL is a DURACELL Inc. trademark.

ULTRALIFE is a ULTRALIFE Batteries, Inc. trademark.

LEMO is a LEMO USA INC. trademark.

SaniCloth is a PDI trademark.

The products described are covered by one or more of the following U.S. Patent Nos.,

4,559,038; 4,565,542; 4,650,469; 5,181,910; 5,338,157; 5,364,242; 5,485,408;

5,531,697; 5,531,698; 5,538,399; 5,540,561; 5,564,915; 5,567,136; 5,567,119;

5,695,473 (Model 6101 only); other patent(s) pending.

©2002 Deltec, Inc. All rights reserved. Printed in U.S.A. 4/02 19824



**ISO 9001 Registered**

**smiths**

Smiths Medical – a part of Smiths Group plc